



Clearview Exact Strep A

Two rapid one-step tests for the direct detection of
Group A Streptococcal Antigen



Clearview

Clearly different. **Clearly better.**

www.clearview.com

Early detection and intervention of Strep A pharyngitis is particularly important in children, allowing treatment to:

- Shorten the course of illness
- Reduce severity of symptoms
- Reduce likelihood of complications
- Reduce spread to others

Features

■ Results in 5 minutes

■ Only one-step

■ Excellent sensitivity and specificity

■ Clear results

■ In-built procedural control

■ Room temperature storage

■ Positive and negative controls included

Benefits

■ Rapid. Patients can be tested and treated in the same visit

■ Easy to use, minimal training required

■ Confidence that the results are accurate and reliable

■ Simple interpretation

■ Confirmation that test has been performed correctly

■ Ready to use - leaves valuable refrigerator space free

■ Quality control procedure

Distributed by:

Unipath Limited, Bedford MK44 3UP, United Kingdom
Tel: +44 (0)1234 835000 Fax: +44 (0)1234 835009

Strep A detection made simple*

1

Extract the throat swab sample using the reagents provided.

2

Cassette – Cap the extraction tube with the dropper provided and squeeze it all onto the sample well of the device.

Dipstick – Immerse the test strip in the extraction tube.

3

Read the result at five minutes.



Positive Negative Invalid

*See package insert for complete instructions.

Product Information

Cassette Cat No: 505288

Dipstick Cat No: 505289

Kit Size 25 Tests

Shelf Life 18 Months

Storage 2 - 30°C

Sample Type Throat Swab

Time to Result 5 minutes

Sensitivity:

Cassette **95.2%**

Dipstick **95.1%**

Specificity: **99%**

Clearly different.
Clearly better.

www.clearview.com

inverness medical
professional diagnostics