



Clearly different.
Clearly better.



Clearview EASY LH

A reliable one-step ovulation test for the qualitative detection of luteinising hormone (LH) in urine of women
as an aid to identify the LH surge

Detection of the LH surge:

- Is an effective tool in the management of fertility
- Indicates that ovulation will occur in the next 24-36 hours
- Provides couples with better information to time intercourse and improve their chances of conception

Features

Benefits


- | | |
|------------------------------|--|
| ■ Results in 3 minutes | ■ Rapid |
| ■ 2 easy sampling methods | ■ Flexible |
| ■ Proven accuracy of >99% | ■ Reliable |
| ■ Inbuilt procedural control | ■ Confirmation that the test has been performed correctly |
| ■ Just one-step | ■ Easy to use, minimal training required |
| ■ Sealed unit after use | ■ Hygienic |
| ■ Ergonomic | ■ Compact and easy to use |
| ■ Room temperature storage | ■ Ready to use, leaves valuable refrigerator space free |
| ■ Long shelf life | ■ Small volume users need not worry that product will expire before used |

Clearview EASY LH

LH detection made simple

1 Use one of the following 2 sampling procedures.

A Dip

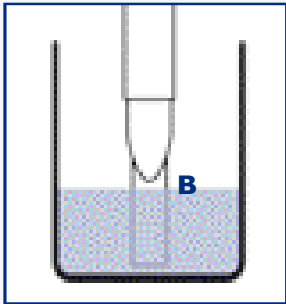


- Dip absorbent sampler into sample to point B for 15 seconds.
- Place cap over absorbent sampler, without inverting device.

B Urine Stream



- Absorbent sampler should be held in urine stream for 3 to 7 seconds **only**.
- Place cap over absorbent sampler, without inverting device.

2 Read result after 3 minutes. A blue Reference Line must appear within 3 minutes otherwise test must be repeated. Any result that appears after 10 minutes must be ignored.

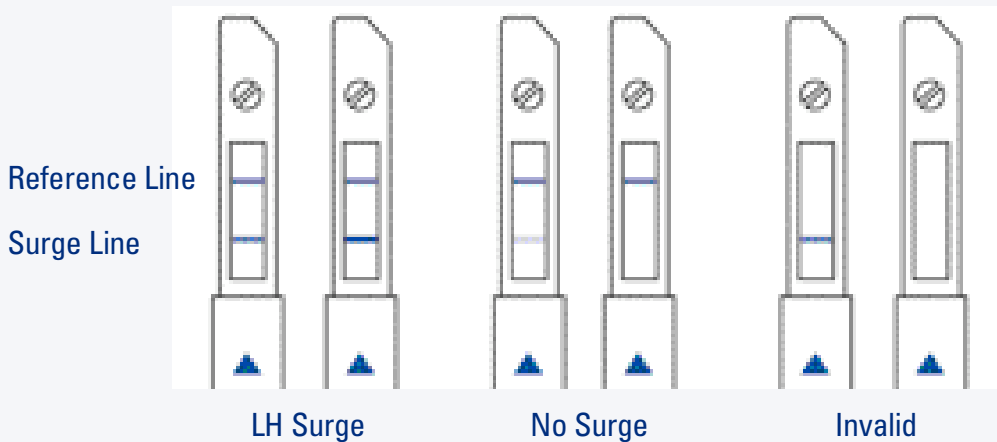

See package insert for complete instructions.

Distributed by:

Unipath Limited, Bedford MK44 3UP, United Kingdom
Tel: +44 (0)1234 835000 Fax: +44 (0)1234 835009

Product Information

Cat No: 505397
(English, Spanish, Italian
French, German)

Cat No: 504436
(Non EU Countries)

Kit Size 20 Tests
(Non EU Countries)

25 Tests
(EU Countries)

Shelf Life 24 Months

Storage 2 - 30°C

Sample Type Urine

Time to Result 3 minutes

2 Sample Dip
Procedures Urine stream

Sensitivity $\geq 40\text{mIU/ml}$
Detects LH Concentrations

Specificity 100%

Accuracy 99%


Clearview

Clearly different.
Clearly better.

www.clearview.com

 **inverness medical**
professional diagnostics